

Deluxe Galvanic High-Frequency Table Top Unit



Spa & Equipment SKINACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.



Toll Free: 1.800.418.0830


www.skinact.com
www.spaandequipment.com

International: 818.834.6640

Measurements



Ease of use Features

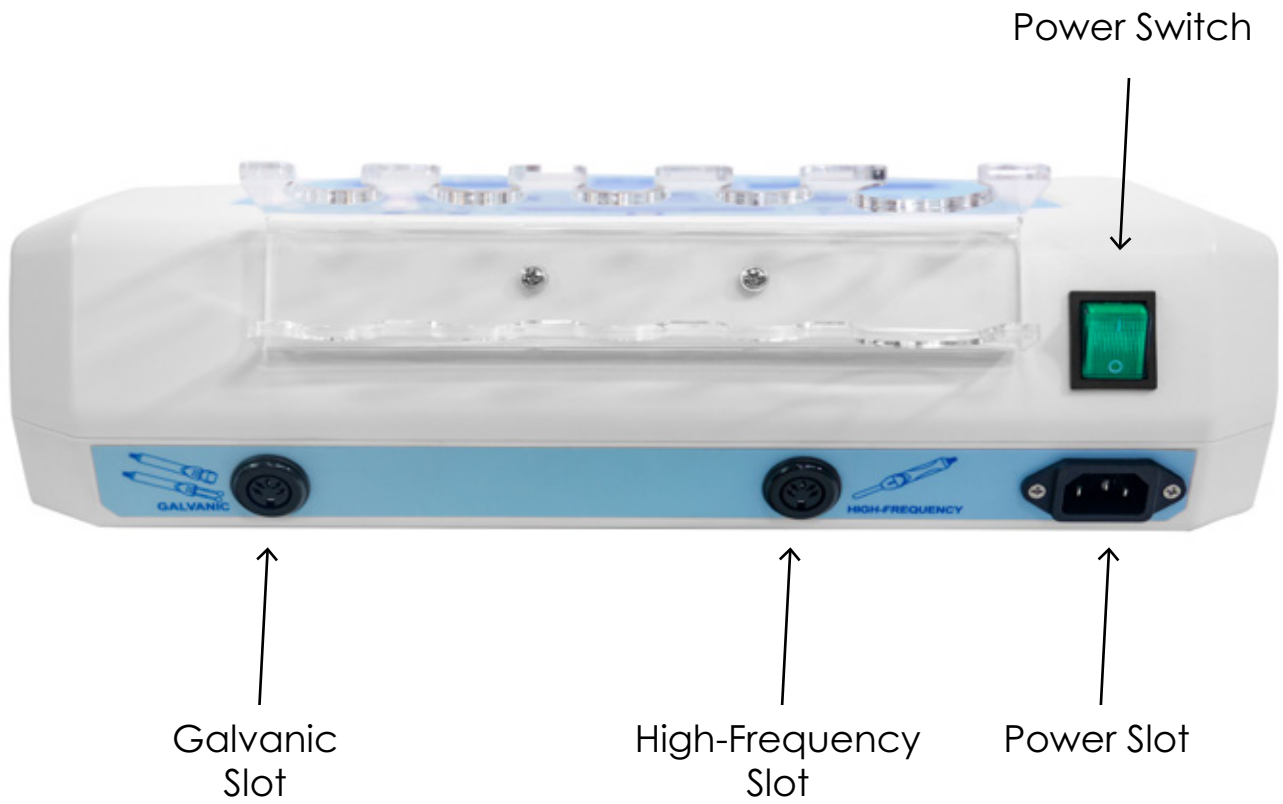

Highlights

High Frequency - High Frequency treatment is a popular treatment method known for its germicidal decongesting, energizing , and stimulating actions. Because of the rapid oscillation of the frequency there are no muscular contractions. The physiological effects are either stimulating or soothing, depending on the method of application. All treatments given with High Frequency should be started with mild current and gradually increased to the required strength. The length of the treatment depends upon the condition to be treated. For general facial treatment, approximately three to five minutes should be adequate.

Galvanic – Galvanic treatment has two functions: the first, iontophoresis promotes deep skin cleansing and the second, iontophoresis facilitates cream, lotion, and solution penetration deep into the skin.

Iontophoresis: This function facilitates product penetration, a process based on the process at the poles, like magnets. While using this Galvanic Current, there are two (2) Poles, Positive and Negative: Positive Pole: Hardens and Firms Tissue - Soothes Nerves - Reduces Redness - Germicidal Negative Pole: Softens and Relaxes Tissue - Irritates Follicles and Stimulates Cells - Increases Blood Supply - Germicidal. Use with any conductive gel and Skin Serum for excellent result.

Specifications:

Length: 15"

Width: 9.75"

HEIGHT: 6"

20W

50/60Hz

100-120V

1-YEAR WARRANTY

SKINACT

Spa & Equipment