

SKIN+ACT



Galvanic Facial Skin Care Unit

SKU25357

Contraindications

Pacemaker (any electrical or rhythmic heart-regulating device)

Pregnancy

Thrombosis/Phlebitis

Skin diseases

Operations (within the last 6 months)

Silicone implants

Hypersensitive or vascular skin

Package Contents


The image shows various components, likely including:

The main Galvanic device itself

Power supply or adapter

Accessories (possibly probes or applicators)

Cables for connections

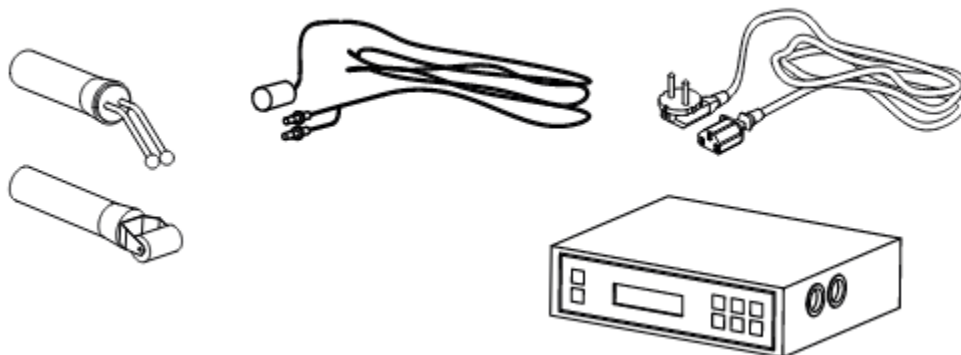

Front Panel



Instructions of Use

Steps are outlined:

Connect the power supply via the rear panel.

Connect the accessories properly in the lateral panel.

Before Treatment

- The treatment area must be clean and dry.
- Ensure both handpieces are in contact with the patient's skin to complete the circuit.
- The polarity of the probes can be changed using the "LEAD" button.
- Cover the roller/ball with a cotton piece or sponge dipped in a suitable lotion.

Starting Treatment

- Choose the desired probes (roller or ball) and connect two identical ones.
- Turn on the unit using the "POWER" button.
- Set the desired power level using "ENERGY" and time using "TIME."
- Press "START/PAUSE" to begin.

Cautions

- Remove jewelry and confirm no metal implants.
- Pregnant women, people with sensitive/inflamed skin, or blood capillary expansions should avoid the treatment.
- Those with heart disease, infectious diseases, hyper-allergic or hyper-sensitive skin should consult a doctor before use.

- Ensure the skin is moistened with plenty of conductive product.

FAQ

- If the rollers fail to rotate, dismantle and clean them. If the issue persists, contact your sales agent.

Maintenance

- Avoid damaging the probe rollers. Do not use abrasive substances for cleaning.
- After treatment, clean, sterilize, and dry all metal roller parts. Store them in the accessory holder.

Technical Sheet

Weights and sizes	Product size (cm)	20.5×22×7
	Gross weight (kg)	2.7
Electrical characteristics	Rated voltage	AC 100-240V
	Frecuency	50/60Hz
	Rated power	4 W

1-800-418-0830 TOLL FREE

1-818-834-6640 INTERNATIONAL

Address: 9667 Canoga Ave, Chatsworth, CA 91311

SKIN+ACT