

Dyno Spa Facial Multi-Function Skincare Machine Specifications Sheet



Spa & Equipment SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.




Toll Free: 1.800.418.0830


www.skinact.com
www.spaandequipment.com

International: 818.834.6640


Measurements

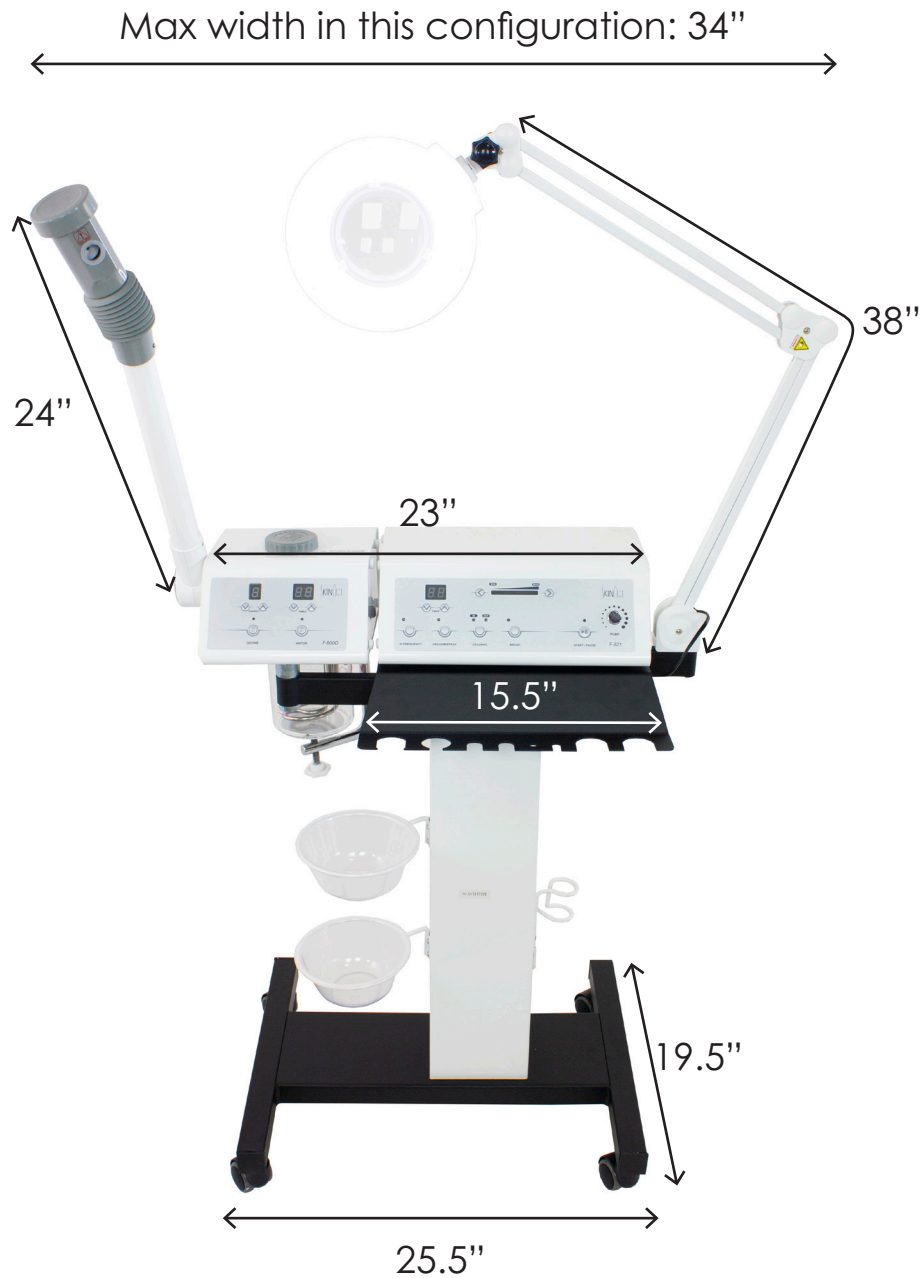

Measurements

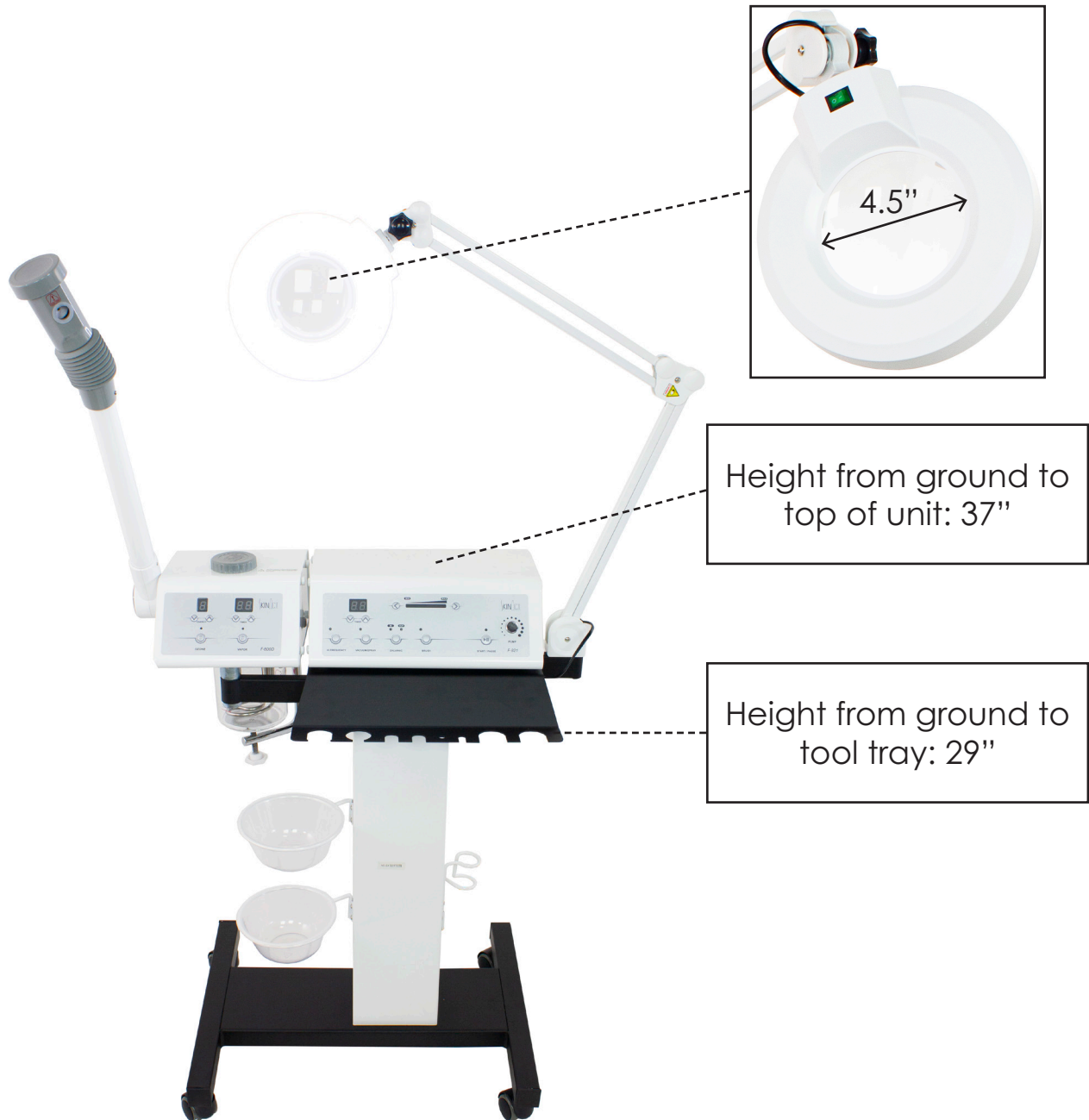

Tools

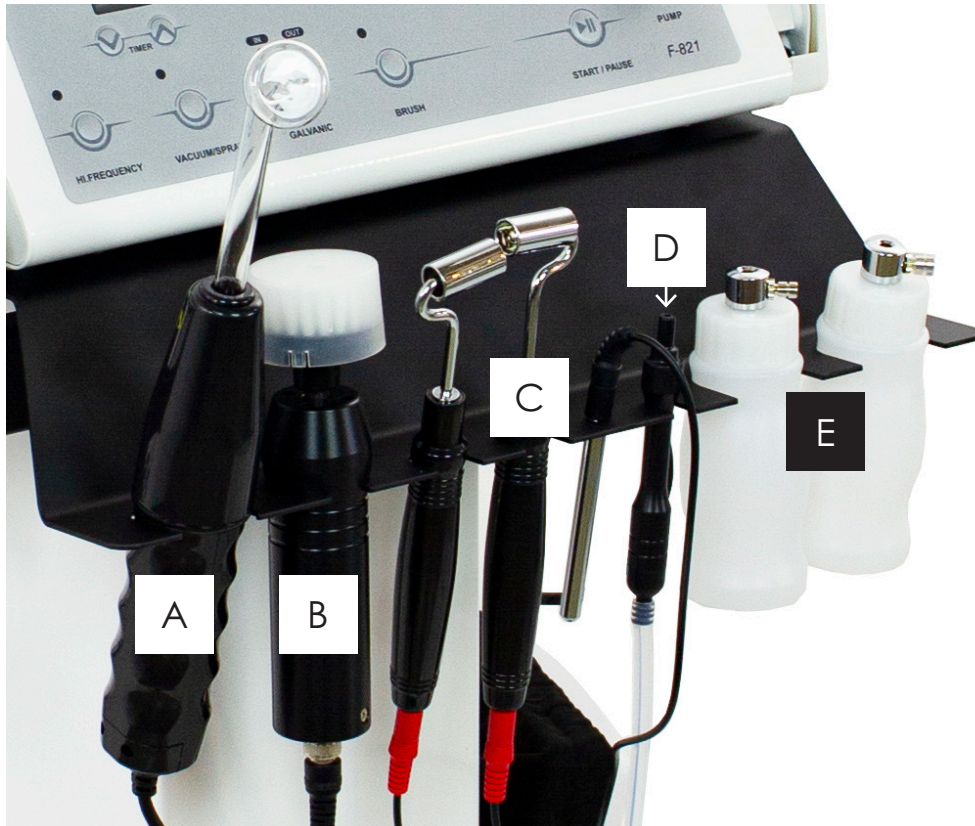

A-High Frequency: Helps treat acne, stimulate circulation, increase glandular activity, and kill germs.

B-Facial Brush: Helps to clean pores deeply and remove dirt/debris.

C-Galvanic: Aids with improving circulation and with product penetration.

D-Vacuum: Helps to deep clean the skin's pores.

E-Spray: Helps to soothe and cool the skin. Also helps with product penetration.

Tools



F-Towel Basins: Holds tools, towels, and any other small essentials.

G-Wood's Lamp: Deep violet rays help expose skin conditions that are not visible to the naked eye.

H-Mag Lamp: Helps to make observations much easier: 5x magnification lens and a soft tube light.

I-Ozone Steamer: Helps moisten and cleanse the skin. Ozone helps increase cell oxygenation.

Highlights

Dimensions/Specifications:

- Height from ground to top of unit: 37"
- Width from steamer arm to magnifying lamp arm: 34"
- BASE ONLY Dimensions: 25.5"L x 19.5"W
- Weight: 72 lbs

Features:

- Vacuum
- Spray
- High Frequency
- Facial Brush
- Magnifying Lamp
- Facial Ozone Steamer
- Galvanic
- Towel / Tool Basins
- Woods Lamp

Warranty Information:

- 1-year warranty

Disclaimer

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility.

All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Equipment Features

This series are composed of metal shell , stainless steel base, fashionable glass water cup, copper spray pipe and so forth. Therefore, it is featured with solid structure, durability and good shape. The special stainless steel base guarantees the stability and safety in high temperature, and it will not result in oxidation or re-composition in high temperature, so as to avoid some oxidized and heavy metal atoms from some traditional material in high temperature when they are at work, to harm the skin. Hence, it can guarantee the requirement for cleanness of vapor in beauty treatment.

The spray head can rotate at various angles, which make the operation more easily.

Apart from the basic herbs function, it is also equipped with time pre-set, five levels spray adjustment, and an automatic water level detection.

Besides, it also equipped with maximum and minimum water level alarming system, so as to prevent the damage for heater parts when the water level is too low, or prevent the users from opening the steamer due to their carelessness when there is no water in the water cup.

Warning

- Please make sure the power line is cut off when there is no water or a lack of water. To avoid any danger, disconnect the power when the water level is low. Otherwise, the users should take full responsibility if there is any accident.
- Please read the detailed instructions carefully or operate the instrument with the help of professional technicians.
- Children shall not use this instrument and get close to or play this instrument within an hour after the instrument stop operating.
- Since this instrument works with high temperature, keep an eye on the instrument when it is working to prevent any accidents or fire.
- Do not use any greasy elements into the water container or the herbs filter mesh, otherwise, water may spray out.
- Do not spray vapor directly down to the client's face too closely, or it will cause scald.
- Do not use the steamer under humid environment, like bathroom, etc.
- Do not use the steamer under the sun for long time, otherwise, it will cause some instabilities.
- Do not conduct electricity between probes directly, or it will lead to elements damages.
- Do not open up the machine by oneself. If there is anything unusual, please consult the sales agent.