

# Quad 4 in 1 Steamer Mag Light High Frequency and Galvanic Machine (SKU24092)



Spa & Equipment SKIN+ACT

Toll Free: 1.800.418.0830

[www.skinact.com](http://www.skinact.com)  
[www.spaandequipment.com](http://www.spaandequipment.com)

International: 818.834.6640

## NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

Spa & Equipment SKIN+ACT




Toll Free: 1.800.418.0830

[www.skinact.com](http://www.skinact.com)  
[www.spaandequipment.com](http://www.spaandequipment.com)

International: 818.834.6640

# Measurements

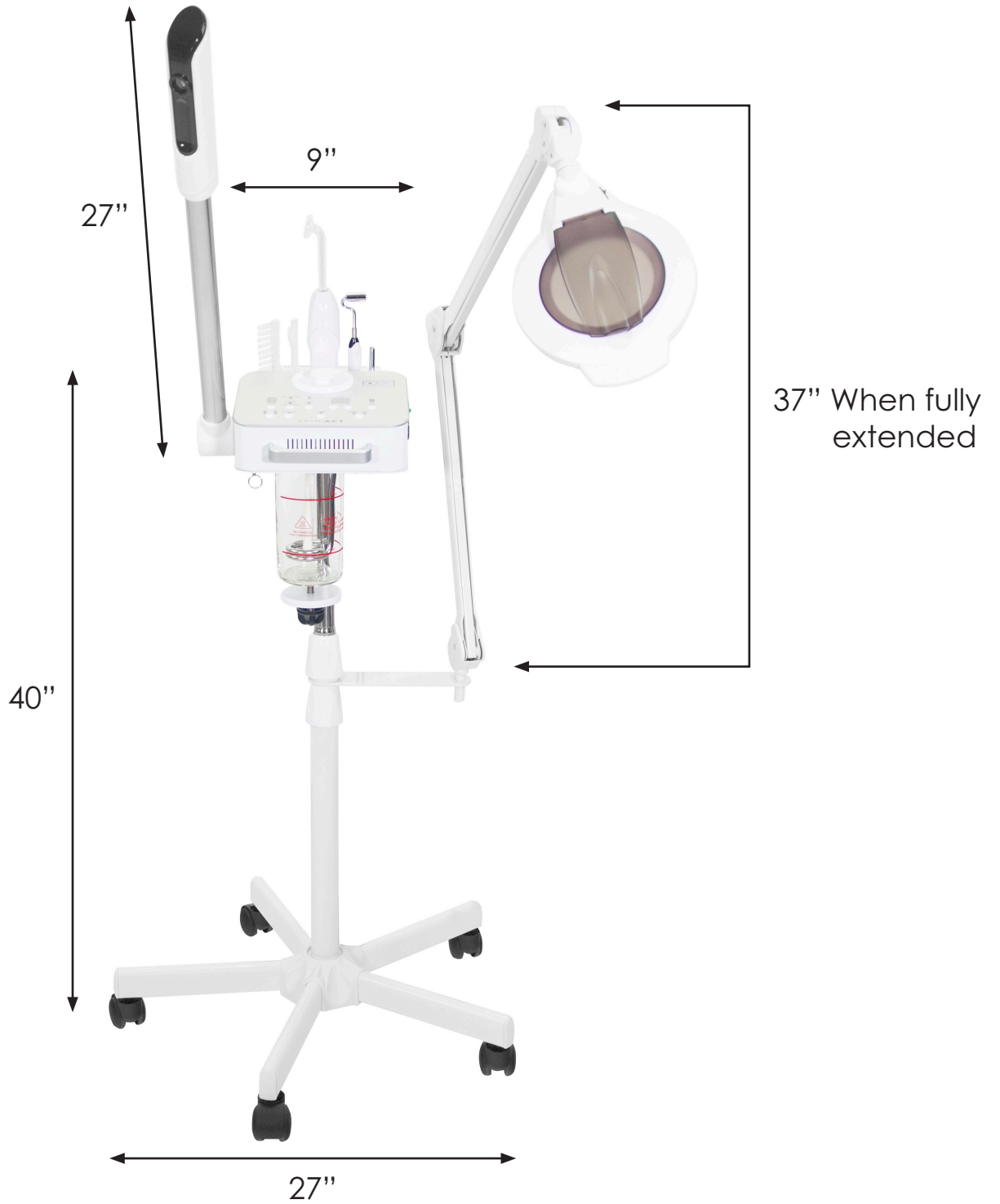

# Tools



**Facial Ozone Steamer:** Moistens and cleanses skin. Steam causes pores to expand and stimulates blood circulation. Comes with an adjustable extension arm that will allow a broader facial steam. Unit Includes a timer, and Alarm Settings. Benefits: Moistens and cleanses the skin. Steam causes pores to expand and stimulates blood circulation. The ozone variety of steam is also capable of increasing cell oxygenation.



**Magnifying LED Light:** Magnifying lamp consists of optical lens and soft light of cold lamp tube. It can make observation cleaner, so acne and dark spots are more easily to be found. It is very suitable treatments and skin analysis.



**High Frequency:** This advanced technology uses powerful electrical current to encourage oxygen circulation in the skin, pushing out toxins and promoting a healthier complexion. Its anti-aging properties stimulate collagen and elastin production, providing a more firm and youthful appearance.



**Galvanic:** Galvanic technology is the latest advance in skin care treatments. It uses direct electrical current to deliver hydration and nutrients directly to your skin by using negative and positive ion currents. By promoting absorption into the surface layer, you will experience maximum results.

# Highlights

This revolutionary equipment features a steamer, magnifying light, high frequency and galvanic capabilities all in one convenient device. With its versatile design, this powerful piece of equipment offers multiple services in one machine. For professional aestheticians or beauty centers looking for an efficient and cost effective device that offers multiple services, the Quad 4 in 1 is the ideal choice as it does it all! Experience convenience on a whole new level with revolutionary Quad 4 in 1!

## Features:

- **Facial Ozone Steamer:** Steam intensity, timer and ozone option.
- **Magnifying LED Light:** Adjustable arm, bright cool-tone LED, and magnifying lens.
- **High Frequency:** 5 professional glass electrodes, high frequency handle, intensity level adjustment.
- **Galvanic:** Negative and positive ion currents, intensity adjustment level, 1 galvanic roller and 1 galvanic rod.

## Specifications:

- Glass beaker: 35 oz
- **1-year limited warranty**

## Disclaimer

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility.

All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

## Equipment Features

This series are composed of metal shell , stainless steel base, fashionable glass water cup, copper spray pipe and so forth. Therefore, it is featured with solid structure, durability and good shape. The special stainless steel base guarantees the stability and safety in high temperature, and it will not result in oxidation or re-composition in high temperature, so as to avoid some oxidized and heavy metal atoms from some traditional material in high temperature when they are at work, to harm the skin. Hence, it can guarantee the requirement for cleanness of vapor in beauty treatment.

The spray head can rotate at various angles, which make the operation more easily.

Apart from the basic herbs function, it is also equipped with time pre-set, five levels spray adjustment, and an automatic water level detection.

Besides, it also equipped with maximum and minimum water level alarming system, so as to prevent the damage for heater parts when the water level is too low, or prevent the users from opening the steamer due to their carelessness when there is no water in the water cup.

## Warning

- Please make sure the power line is cut off when there is no water or a lack of water. To avoid any danger, disconnect the power when the water level is low. Otherwise, the users should take full responsibility if there is any accident.
- Please read the detailed instructions carefully or operate the instrument with the help of professional technicians.
- Children shall not use this instrument and get close to or play this instrument within an hour after the instrument stop operating.
- Since this instrument works with high temperature, keep an eye on the instrument when it is working to prevent any accidents or fire.
- Do not use any greasy elements into the water container or the herbs filter mesh, otherwise, water may spray out.
- Do not spray vapor directly down to the client's face too closely, or it will cause scald.
- Do not use the steamer under humid environment, like bathroom, etc.
- Do not use the steamer under the sun for long time, otherwise, it will cause some instabilities.
- Do not conduct electricity between probes directly, or it will lead to elements damages.
- Do not open up the machine by oneself. If there is anything unusual, please consult the sales agent.