

**Supra 5 in 1 Microdermabrasion +
Ultrasonic + LED Light Therapy +
Microcurrent + Cold & Hot Therapy**
(SKU17813)



Spa & Equipment SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.

Spa & Equipment SKIN+ACT



Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

Tools



Diamond Microdermabrasion: Diamond microdermabrasion resurfaces the skin non-surgically by using sterile diamond heads to peel and rub off the dead cells of the top skin layer. The microdermabrasion handles are attached to a vacuum. So, the dead skin cells are sucked away.



Ultrasonic: Stimulates the skin tissue with acoustic vibrations. Improves cell membrane and regenerates collagen. Reduces wrinkles and fine lines for younger looking and feeling skin. Ultrasonic also generates frictional heat which accelerates blood circulation and helps muscles to relax. Decomposes cellulite and allows deeper product penetration into the skin.



LED Light Therapy: The Red, Green, and Blue LED light therapy provides several benefits such as: the reduction of fine lines/ wrinkles, the reduction of acne, as well as lightening of hyper pigmentation spots on the face and neck area. Each LED light color has it's own treatment benefits.



Microcurrent: The Microcurrent stimulates the muscle groups that need help lifting and firming. The electrical current increases the ATP and collagen production in the skin for younger, beautiful results without any down time. Microcurrent is a safe and painless facial which helps erase fine lines and wrinkles.



Cold/Hot Hammer: The Cold/Hot Hammer cools and soothes the skin with the cold side and warms the skin on the other side. Cooling the skin is important after a microdermabrasion treatment. Warming the skin opens pores and helps products to penetrate more deeply.

Highlights

This revolutionary equipment features a diamond microdermabrasion, ultrasonic, LED light therapy, microcurrent, and cold/hot hammer treatment capabilities all in one convenient device. With its versatile design, this powerful piece of equipment offers five services in one machine. For professional aestheticians or beauty centers looking for an efficient and cost effective device that offers multiple services, the Supra 5 in 1 Microdermabrasion + Ultrasonic + LED Light Therapy + Microcurrent + Cold & Hot Therapy is the ideal choice as it does it all!

Features:

- **Diamond Microdermabrasion:** Steam intensity, timer and ozone option.
- **Ultrasonic:** 5 professional glass electrodes, high frequency handle, intensity level adjustment.
- **LED Light Therapy:** 3 LED light treatments (red, green, and blue).
- **Microcurrent:** Microcurrent treatment wand.
- **Cold/Hot Hammer Treatment:** Adjustable arm, bright cool-tone LED, and magnifying lens.

Specifications:

Diamond Microdermabrasion

Diamond Microdermabrasion has 3 tip grades

3 Large: Fine, Medium, Coarse

3 Medium: Fine, Medium, Coarse

3 Small: Fine, Medium, Coarse

The optional Diamond Microdermabrasion comes with 9 Tips and 3 Handles with filters

The Diamond Microdermabrasion on this unit is class 1 grade. It is only for the surface of the skin

Ultrasonic

This kit comes with two different ultrasonic handles. The bigger flat handle is for the larger areas like the back or stomach. The smaller probe is for around the eyes.

LED Light Therapy

Red: The red light therapy stimulates collagen production that is responsible for the elasticity and firmness of your skin. Collagen is essential to reduce fine lines and wrinkles.

Blue: The blue light therapy kills the Propionibacterium acnes bacteria. This bacteria is one of the causes of acne. The bacteria absorbs the light and the light kills the bacteria.

Green: The green LED treatment will lighten hyper pigmentation spots on the face and neck area.

Highlights

Microcurrent

- Tones and condition facial muscles, while tightening skin and improving elasticity
- Smooth laughter lines, wrinkles, fine lines and improve skin complexion
- Boost circulation and increase supplies of nutrients to skin
- Body sculpting and toning
- Lymphatic drainage
- Minimize eye bags and dark circles

Cold/Hot Hammer

Cold Hammer Minimum Temperature: ~52° F / ~11° C

Hot Hammer Minimum Temperature: ~120° F / ~49° C

- **1-year limited warranty**