Modena 19 in 1 Skin Care System Specifications Sheet





Toll Free: 1.800.418.0830 www.skinact.com www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.











Toll Free: 1.800.418.0830

www.skinact.com www.spaandequipment.com

International: 818.834.6640

Measurements





Measurements







A-Facial Ozone Steamer: Helps to moisten and cleanse the skin. Ozone option helps with cell oxygenation. Digital timer and adjustable intensity.

B-Magnifying Lamp: Helps to make observations much easier. There are a 5x magnification lens and a soft tube light.

C-Sterilizer and Hot Towel Cabi: The sterilizer helps to sterilize electrodes, scissors, glass, brushes, and more. Hot towel cabi lets you warm towels for your clients.





- **D-Vacuum:** Helps to deep clean the skin and bring impurities to the surface. 3 different shaped tips are included.
- **E-Spray:** Helps to soothe and cool the skin. Also helps with product penetration. 2 spray bottles are included.
- **F-Hot Cold Hammer:** Cold helps with cooling the skin after a dermabrasion treatment. Hot helps with accelerating the absorption of nutrients into the skin.





G-Galvanic: Aids with improving circulation and with product penetration. A negative probe, rolling probe, and ball probe are included.

H-Ultrasonic: Acoustic vibrations help stimulate skin tissue, decompose cellulite, and accelerate circulation. 2 ultrasonic handles are included.

I-Skin Scrubber: Vibrates up to 28,000 times per second. Helps to reduce wrinkles. Also helps remove dark spots, dead skin, and more.





- **J-High Frequency:** Helps to stimulate circulation, increase grandular activity, treat acne, kill germs, and more. Comes with 3 different electrodes.
- **K-Facial Brush:** Helps to deep clean pores and remove dirt/debris. Lightweight and quiet. Comes with 5 different interchangable brush heads.
- **L-Wood's Lamp:** Uses deep violet rays to expose skin conditions that are not visible in regular light. Different skin conditions appear as varied violet shades.





M-Magic Gloves: Used to provide microcurrent facial treatments. An added bonus is that you can provide a gentle massage at the same time.

N-Spot Remover: Helps to shrink or eliminate skin tags, acne pimples, minor fibromas, and cherry angiomas.

O-*Optional Diamond Microdermabrasion: Leaves the skin feeling soft and smooth. 3 handles and 9 tips included. The tips have 3 grades of coarseness.





P-Metal Racks: Versatile racks for holding towels, tools, disposables, products, and any other small items.

Q-Towel/Tool Basins: Basins for holding towels, disposables, products, or anything else.



Highlights

Dimensions/Specifications:

• Height from ground to top of unit: 44.5"

• Width: 29"

• BASE ONLY Dimensions: 16.5"L x 16.5"W

• Weight: 121 lbs

Features:

• Facial Ozone Steamer

- Magnifying Lamp
- Aroma/Essential Oil Therapy
- Vacuum
- Spray
- Hot / Cold Hammer
- Galvanic
- Ultrasonic
- Skin Scrubber
- High Frequency
- Facial Brush
- Metal racks
- Towel / Tool Basins
- Woods Lamp
- Hot Towel Cabi
- UV Sterilizer
- Spot Remover
- Magic Gloves

Warranty Information:

• 1-year warranty



Disclaimer

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility.

All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Equipment Features

This series are composed of metal shell, stainless steel base, fashionable glass water cup, copper spray pipe and so forth. Therefore, it is featured with solid structure, durability and good shape. The special stainless steel base guarantees the stability and safety in high temperature, and it will not result in oxidation or re-composition in high temperature, so as to avoid some oxidized and heavy metal atoms from some traditional material in high temperature when they are at work, to harm the skin. Hence, it can guarantee the requirement for cleanness of vapor in beauty treatment.

The spray head can rotate at various angles, which make the operation more easily. Apart from the basic herbs function, it is also equipped with time pre-set, five levels spray adjustment, and an automatic water level detection.

Besides, it also equipped with maximum and minimum water level alarming system, so as to prevent the damage for heater parts when the water level is too low, or prevent the users from opening the steamer due to their carelessness when there is no water in the water cup.

Warning

- Please make sure the power line is cut off when there is no water or a lack of water. To
 avoid any danger, disconnect the power when the water level is low. Otherwise, the
 users should take full responsibility if there is any accident.
- Please read the detailed instructions carefully or operate the instrument with the help of professional technicians.
- Children shall not use this instrument and get close to or play this instrument within an hour after the instrument stop operating.
- Since this instrument works with high temperature, keep an eye on the instrument when it is working to prevent any accidents or fire.
- Do not use any greasy elements into the water container or the herbs filter mesh, otherwise, water may spray out.
- Do not spray vapor directly down to the client's face two closely, or it will cause scald.
- Do not use the steamer under humid environment, like bathroom, etc.
- Do not use the steamer under the sun for long time, otherwise, it will cause some instabilities.
- Do not conduct electricity between probes directly, or it will lead to elements damages.
- Do not open up the machine by oneself. If there is anything unusual, please consult the sales agent.

