Platinum Multifunction Skin Care System (SKU18263)





Toll Free: 1.800.418.0830 www.skinact.com www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.












Toll Free: 1.800.418.0830

www.skinact.com www.spaandequipment.com

International: 818.834.6640

Measurements

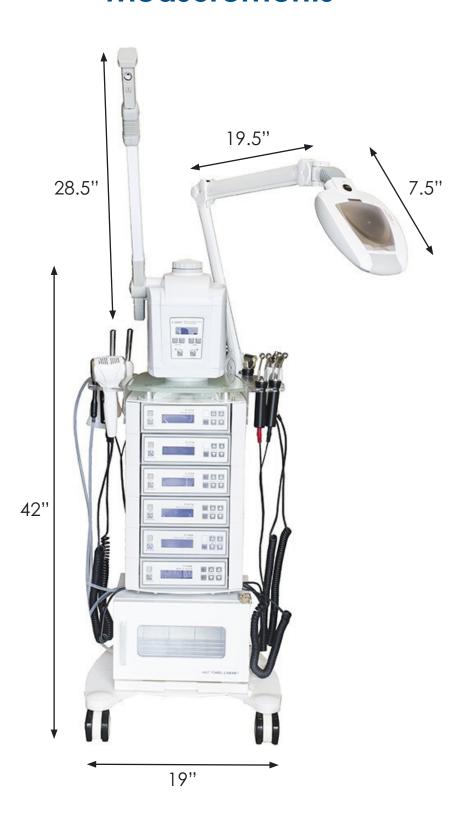



Measurements





Tools

Standard Features – These come standard with every Platinum Multifunction SkinCare System



A: Facial Ozone Steamer

Comes with an adjustable extension arm that allows a broader facial steam. Unit includes a timer and alarm settings as well as an optional ozone setting.

Benefits: Moistens and cleanses the skin. Steam causes pores to expand and stimulates blood circulation. When the optional ozone setting is active, the steamer helps to increase cell oxygenation.

B: Magnifying Lamp

Consists of an optical lens and a soft tube light which is cool to the touch. Very suitable for treatments and skin analysis. It makes observation much easier, so acne and dark spots are found more quickly.

C: Hot Towel Cabi

Great for warming towels for your clients during facials.



Tools

Attention: Below are the tools which you can choose from for this equipment. Total of 6 Treatments out of 9 options available.



High Frequency: Stimulates circulation of the blood, increases glandular activity, aids in elimination and absorption, increases metabolism, kills germs and bacteria, generates heat inside the tissue, and helps products to go deeper into the skin. Benefits: Aids acne treatment. Also kills bacteria and possible skin infections.



Microcurrent with Biolift: Stimulates the muscle groups that need lifting and firming. The electrical current increases ATP and collagen production in the skin for younger, more vibrant looking skin.



Facial Brush: Rotating brush system which includes a variety of attachments for different purposes. Lightweight for easy handling. Quiet operation and easy to access. Benefits: Cleanses pores deeply, removing any dirt and debris.



Hot / Cold Hammer: The cold side of the hammer is designed for cooling down the skin after a dermabrasion treatment; it will reduce redness of the skin. The hot side is designed for accelerating the absorption of nutrients into the skin.



Tools



Ultrasonic: Stimulates the skin tissue with acoustic vibrations. Improves cell membrane and causes collagen regeneration. Reduces wrinkles and fine lines for younger looking and feeling skin. Ultrasonic also generates frictional heat which accelerates blood circulation and helps muscles to relax. Decomposes cellulite and allows deeper product penetration into the skin. Benefits: Utilizes sonic vibrations to reduce lines, leaving the skin firm and smooth. Features an intensity control.



Vacuum and Spray: Vacuum allows a deep cleansing of the skin. Brings skins impurities to the surface, which helps with easy extractions. Spray cools and soothes the skin. It also helps with product absorption into the skin.



Galvanic: Galvanic current contains negative ions while treatment solutions contain positive ions. The current pushes products deeper into the skin. Products neutralize toxins, balance pH, and kill harmful free radicals. Galvanic also accelerates blood circulation. Benefits: Improves blood circulation and facilitates deeper product penetration.



Diamond Microdermabrasion: Removes dead skin cells while exfoliating. Skin feels softer and smoother immediately following treatment.



Skin Scrubber: Helps to reduce wrinkles. It also helps to remove dark spots, dead skin, and other skin problems.



Highlights

The Platinum Multifunction SkinCare System is perfect for those who want a custom machine. It comes with 3 standard tools and 6 tools that you get to choose.

Standard Features – These come standard with every Platinum Multifunction SkinCare System:

Facial Ozone Steamer - Comes with an adjustable extension arm that allows a broader facial steam. Unit includes a timer and alarm settings as well as an optional ozone setting.

Benefits: Moistens and cleanses the skin. Steam causes pores to expand and stimulates blood circulation. When the optional ozone setting is active, the steamer helps to increase cell oxygenation.

Magnifying Lamp - Consists of an optical lens and a soft tube light which is cool to the touch. Very suitable for treatments and skin analysis. It makes observation much easier, so acne and dark spots are found more quickly.

Hot Towel Cabi - Great for warming towels for your clients during facials.

Attention: Below are the tools which you can choose from for this equipment. Please choose 6:

Vacuum and Spray - Vacuum allows a deep cleansing of the skin. Brings skins impurities to the surface, which helps with easy extractions. Spray cools and soothes the skin. It also helps with product absorption into the skin.

High Frequency - Stimulates circulation of the blood, increases glandular activity, aids in elimination and absorption, increases metabolism, kills germs and bacteria, generates heat inside the tissue, and helps products to go deeper into the skin. Benefits: Aids acne treatment. Also kills bacteria and possible skin infections.

Facial Brush - Rotating brush system which includes a variety of attachments for different purposes. Lightweight for easy handling. Quiet operation and easy to access. Benefits: Cleanses pores deeply, removing any dirt and debris.

Galvanic – Galvanic current contains negative ions while treatment solutions contain positive ions. The current pushes products deeper into the skin. Products neutralize toxins, balance pH, and kill harmful free radicals. Galvanic also accelerates blood circulation. Benefits: Improves blood circulation and facilitates deeper product penetration.

Microcurrent with Biolift - Stimulates the muscle groups that need lifting and firming. The electrical current increases ATP and collagen production in the skin for younger, more vibrant looking skin.

Skin Scrubber - Helps to reduce wrinkles. It also helps to remove dark spots, dead skin, and other skin problems.

Ultrasonic - Stimulates the skin tissue with acoustic vibrations. Improves cell membrane and causes collagen regeneration. Reduces wrinkles and fine lines for younger looking and feeling skin. Ultrasonic also generates frictional heat which accelerates blood circulation and helps muscles to relax. Decomposes cellulite and allows deeper product penetration into the skin. Benefits: Utilizes sonic vibrations to reduce lines, leaving the skin firm and smooth. Features an intensity control.

Hot / Cold Hammer - The cold side of the hammer is designed for cooling down the skin after a dermabrasion treatment; it will reduce redness of the skin. The hot side is designed for accelerating the absorption of nutrients into the skin.

Diamond Microdermabrasion - Removes dead skin cells while exfoliating. Skin feels softer and smoother immediately following treatment

Unit Weight: 96 lbs



Disclaimer

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility.

All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

Equipment Features

This series are composed of metal shell, stainless steel base, fashionable glass water cup, copper spray pipe and so forth. Therefore, it is featured with solid structure, durability and good shape. The special stainless steel base guarantees the stability and safety in high temperature, and it will not result in oxidation or re-composition in high temperature, so as to avoid some oxidized and heavy metal atoms from some traditional material in high temperature when they are at work, to harm the skin. Hence, it can guarantee the requirement for cleanness of vapor in beauty treatment.

The spray head can rotate at various angles, which make the operation more easily. Apart from the basic herbs function, it is also equipped with time pre-set, five levels spray adjustment, and an automatic water level detection.

Besides, it also equipped with maximum and minimum water level alarming system, so as to prevent the damage for heater parts when the water level is too low, or prevent the users from opening the steamer due to their carelessness when there is no water in the water cup.

Warning

- Please make sure the power line is cut off when there is no water or a lack of water. To
 avoid any danger, disconnect the power when the water level is low. Otherwise, the
 users should take full responsibility if there is any accident.
- Please read the detailed instructions carefully or operate the instrument with the help of professional technicians.
- Children shall not use this instrument and get close to or play this instrument within an hour after the instrument stop operating.
- Since this instrument works with high temperature, keep an eye on the instrument when it is working to prevent any accidents or fire.
- Do not use any greasy elements into the water container or the herbs filter mesh, otherwise, water may spray out.
- Do not spray vapor directly down to the client's face two closely, or it will cause scald.
- Do not use the steamer under humid environment, like bathroom, etc.
- Do not use the steamer under the sun for long time, otherwise, it will cause some instabilities.
- Do not conduct electricity between probes directly, or it will lead to elements damages.
- Do not open up the machine by oneself. If there is anything unusual, please consult the sales agent.

