

Supra Crystal Microdermabrasion Machine

(Made in the USA | Lifetime Warranty)

SKU17814



Spa & Equipment SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.



Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

Control Switches & Panel:



fan switch

power outlet
socket



vacuum meter

regulator knob

power switch

Setup:



- Connect the Black Hose (No. 2) to the Waste Crystal Bottle, like the picture shown above.
- Connect the Clear Hose (No. 1) to the Fresh Crystal Bottle, like the picture shown above.
- Connect both Black Hose (No.2) and Clear Hose (No.1) to the Crystal Pen like the picture shown. Pay attention to the position marking on the crystal handle, as wrong connection may lead to lose the suction during treatment.
- Use the Crystal Filling Funnel to fill the fresh crystals into the fresh crystal bottle.



Preparation:

1. Make sure to sterilize your hands and we recommend to wear disposable latex gloves and facemask during the treatment.
2. Make sure your client removes all the jewelry from the area that will be treated.
3. Prepare the skin surface for treatment by first cleansing the area using the prescribed cleanser. You must also cool the skin down by using chilled towels. This will remove any residual cleansing product left on the skin. However make sure to use damp towels NOT wet since water will clog and/or damage the unit.
4. Before starting the treatment make sure the skin is completely dry.
5. Take your crystal pen, making sure it, especially the cap is either properly sterilized or replaced.
6. Start your treatment
7. Once completed the treatment. Make sure to cool the treated surface by applying cold towels or cold surface for about 2 minutes so skin will cool down.
8. Remember the skin has just experienced an abrasive treatment. It is normal that there may be some redness as well as a warm effect even after 2 minutes cool down.
9. We highly recommend to use SkinAct® Oxygen infusion spray With Vitamin C or Brightening after this treatment to help the skin heal faster.
10. Make sure to put sunblock after the treatment since skin is very sensitive to sun after the treatment.
11. Make sure the clients do not use any other abrasive product on the skin at least for next 4 days, as it may cause an adverse reaction.

Caution:

- When not in use, your unit should be stored in a clean dry place with power cord unplugged.
- Never store the unit with crystals in either the new or discard containers. Crystals absorb moisture from the air and can clog the machine, if this is inadvertently done, the machine must thoroughly be cleaned before it is turned on.
- Leave crystal pen connected to the hoses to help prevent humidity from entering the machine.
- Never drop, bump or stack anything on your machine.
- Double check the power supply before connecting to the machine.
- Be sure to unplug the power cord from the power outlet when it comes to cleaning.
- Use a piece of dry cloth for cleaning the device casing.
- Never use a damp cloth to clean the face panel.

Maintenance:

1. Remove the cover for No. 2 Waste Crystal Bottle.
2. Remove the Waste Crystal Filter – Big like the picture A shown by unscrewing.
3. Use Pen Brush provided to brush off and clean the filter as there maybe waste crystal residuals remaining on the filter effecting machine's efficiency.
4. After cleaning, screw back the filter. Make sure it's properly fasten, if not waste crystals may enter to 2nd level filtering system which is located at the back of the unit.
5. Close and fasten the cap tightly, a leak of air may lead to incomplete vacuum stage which means it will alter the suction power directly.



Picture A



Picture B

Specifications:

Power Supply: 120V

Frequency: 60HZ

Power Consumption: 300W

Fuses: 3A

Weight: 36LBS

Dimenstions: 16 x 8 x 13 INCHES

