

Diamond Microdermabrasion Machine

(SKU16141)



Spa & Equipment SKIN+ACT

Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

NOTICE

No part of this document may be photocopied, reproduced, or translated into any other language without prior written consent of Spa and Equipment/SkinAct.

The information in this document is subject to change without notice at any time and should not be construed as a commitment by Spa and Equipment/SkinAct.

Spa and Equipment/SkinAct assumes no responsibility for any errors that may be in this document, nor does it make expressed or implied warranty with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Spa and Equipment/SkinAct shall not be liable for incidental or consequential damages arising out of the furnishing, performance, or use of this document and the program material which it describes.



Toll Free: 1.800.418.0830

www.skinact.com
www.spaandequipment.com

International: 818.834.6640

Measurements



Spa & Equipment SKIN+ACT

Tools



Diamond Microdermabrasion: Diamond microdermabrasion resurfaces the skin non-surgically by using sterile diamond heads to peel and rub off the dead cells of the top skin layer. The microdermabrasion handles are attached to a vacuum. So, the dead skin cells are sucked away.

Highlights

The diamond microdermabrasion (crystal-free) system replaces the micro-crystal flow with a diamond tip wand. Areas closer to the eyes and mouth can be abraded more accurately with diamond abrasion (no risk of stray crystals causing damage or being ingested).

Provides a Non-Surgical skin resurfacing procedure by using sterile diamond heads to peel and rub off the dead cells at the top skin layer followed by vacuuming/suction removing particles along with any dirt and dead skin. This procedure removes skin debris, scars, blemishes, wrinkles and uneven pigmentation of the skin. Moderate usage of diamond dermabrasion accompanied with skin products that penetrate through the top skin layer and reach the dermis level help to replenish natural nutrients and restore cell activity promoting healthy skin.

Dermabrasion treatment is usually a healthy, glowing, beautiful skin.

The Benefits of treatments with our Diamond Microdermabrasion machines:

- Rejuvenation of sun damaged skin - face, neck, shoulders, back, arms, and legs
- Reduction of age spots
- Minimization of blotchy skin coloring
- Reduction of acne and superficial scars from past injury
- Extraction of blackheads and whiteheads
- Reduction of oily skin
- Improvement of overall skin health

Features:

- **Diamond Microdermabrasion:** Steam intensity, timer and ozone option.

Specifications:

Diamond Microdermabrasion

Diamond Microdermabrasion has 3 tip grades

3 Large: Fine, Medium, Coarse

3 Medium: Fine, Medium, Coarse

3 Small: Fine, Medium, Coarse

The optional Diamond Microdermabrasion comes with 9 Tips and 3 Handles with filters

The Diamond Microdermabrasion on this unit is class 1 grade. It is only for the surface of the skin