


Radio Frequency Beauty Instrument

INSTRUCTION

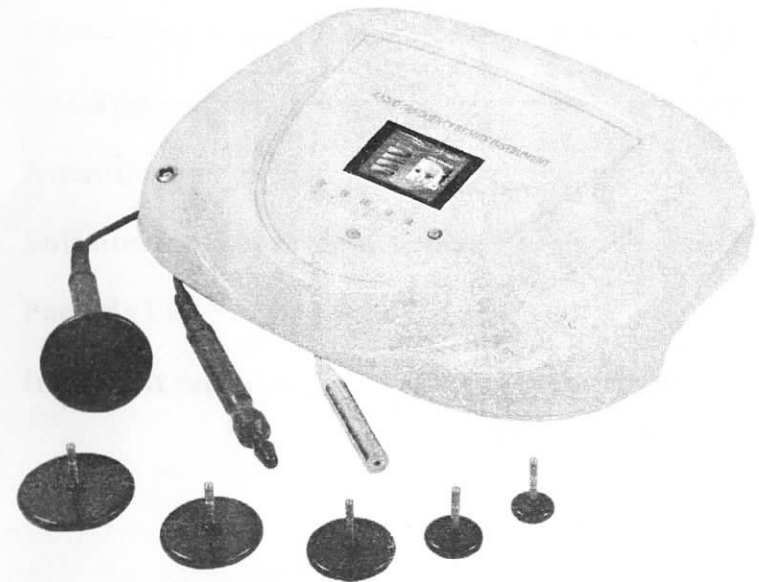

Dear Customer,
Thanks for your choosing our products, for the best capability
and safety, please read the manual before operation.
Please reserve the manual for your reference.

Content

Notice·····	1
Function and Principle·····	2
Comparasion of Before and After·····	3
Brief Introduction·····	4
Control Panel and Acceessories·····	5
Operation guidance·····	6
Attention·····	8
Solution for simple malfunction·····	10
Packing List·····	11
Important safety notice·····	12

Notice

● **Attention:** Before operation, please read below information carefully.

● **Treatment attention:**

1. Select proper treatment head according to specific body part.
2. The intensity should be adjusted from weak to strong, to avoid Sudden strong stimulation that makes the client get unnecessary fear.
3. When make treatment, if the body feel pain or the skin become red and imprint, should stop the treatment at once.
4. The treatment head should move slowly on the surface of skin, don't leave on any body part for too long time.
5. During treatment, it is forbidden to wear any metallic article.

● **Taboo:**

1. It is forbidden to apply treatment to pregnant women.
2. It is forbidden to apply treatment to epileptic and psychopath.
3. If people having fever, infection, skin disease (wheek, tetter, herpes), fresh scar wound, serious chronic disease (cancer, AIDS, hepatitis, blood disease etc.) or cruor disease, heart disease, it is for bidden to be applied the treatment.
4. If people having phlebitis, having medical operation in two months (vein cirrhosis), having varicosity disease or limb trouble, It is forbidden to be applied the treatment.
5. If people taking medicine such as anti inflamed medicine, anticoagulant, antibiotics and so on for a long time, and the skin is easy to be bruise after light impact, please inquiry the doctor before applying the treatment.

Function and Principle

● **skin therapy**

Function: rise and frap skin, whiten and tender skin, control dark Sore, reduce wrinkle, frap loose belly after giving birth.

Principle: Double pole radio frequency beauty function, it employ frequent swtich current. When the current pass through body tissue, it can make effect to the electron and generate the phenomena **polarization electron movement** and make electron impact each other, rotating and twisting, which can generate biology heat energy and increase the

temperature deep inside of the skin. In this way to Promote collagen fibre to frap and reborn collagen albumen, therefore can remove wrinkle, and frap and tender skin.

Curing symptom


1. Crow-feet
2. Blepharoptosis
3. Sagging of face and chin
4. other sin ageing symptom

● **Fat therapy**


Function: Burning fat, slimming body, improve astriction, strengthen elasticity.

Principle: Double pole frequency resonate with body biology electricity, make fat cell break down, frap and release, meanwhile promote lymph system de-toxin and accelerate blood circulation, improve metabolism. **Curing**





symptom:

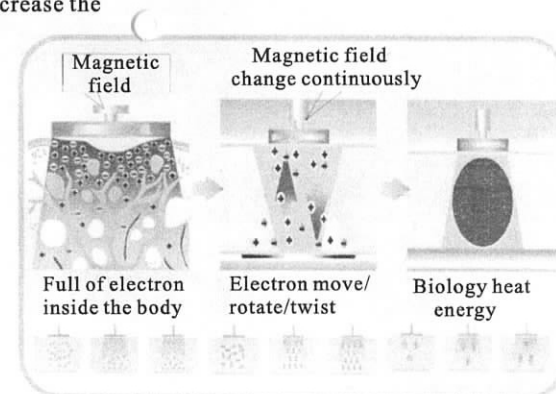

RF polarization working principle picture

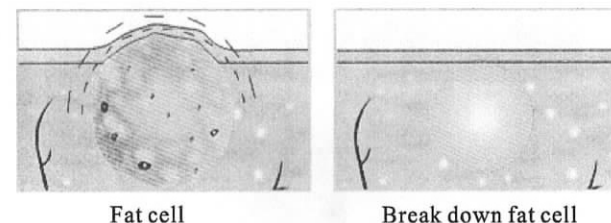

Treatment effect

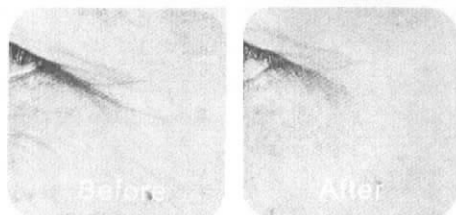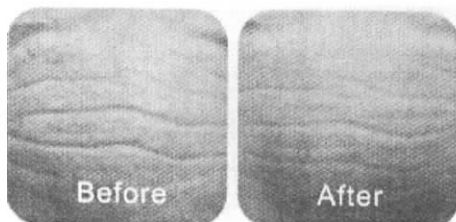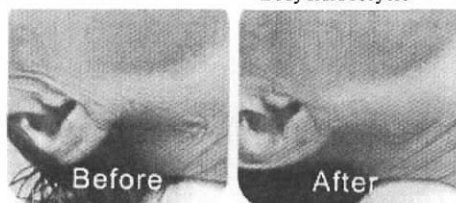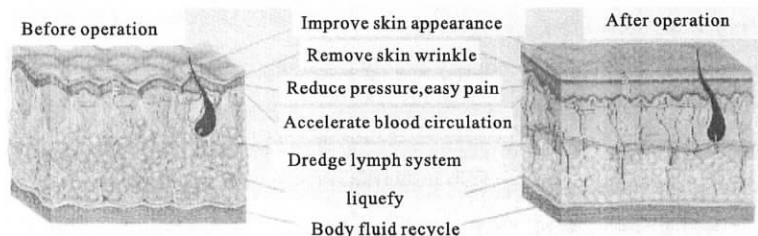


Ten Advantages:

- No operation
- No intrude
- No blood
- No pain
- No scar
- No wound
- No need convalescence
- Short treatment time
- Rapid effect
- Long keep

Brief Introduction

Power Connection

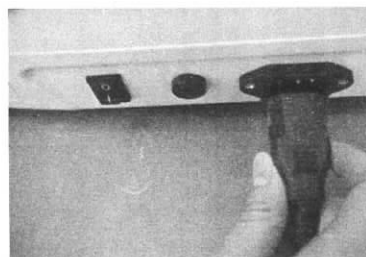


Connect the wire to the back of the mainframe



Connect the plug as picture

Spare parts connection

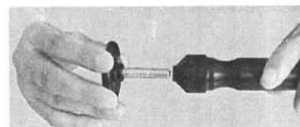

Connect the facial probe to HF handle






Connection as picture



Connect to the mainframe

Control panel and accessories

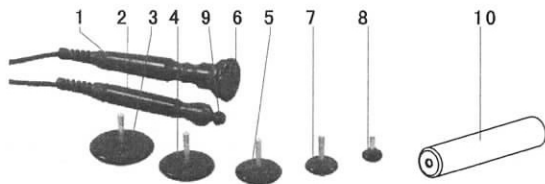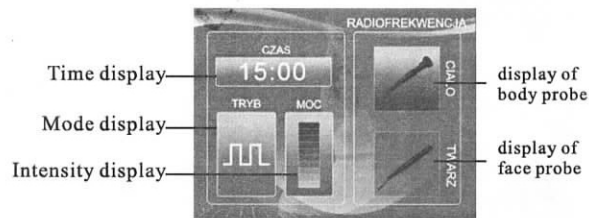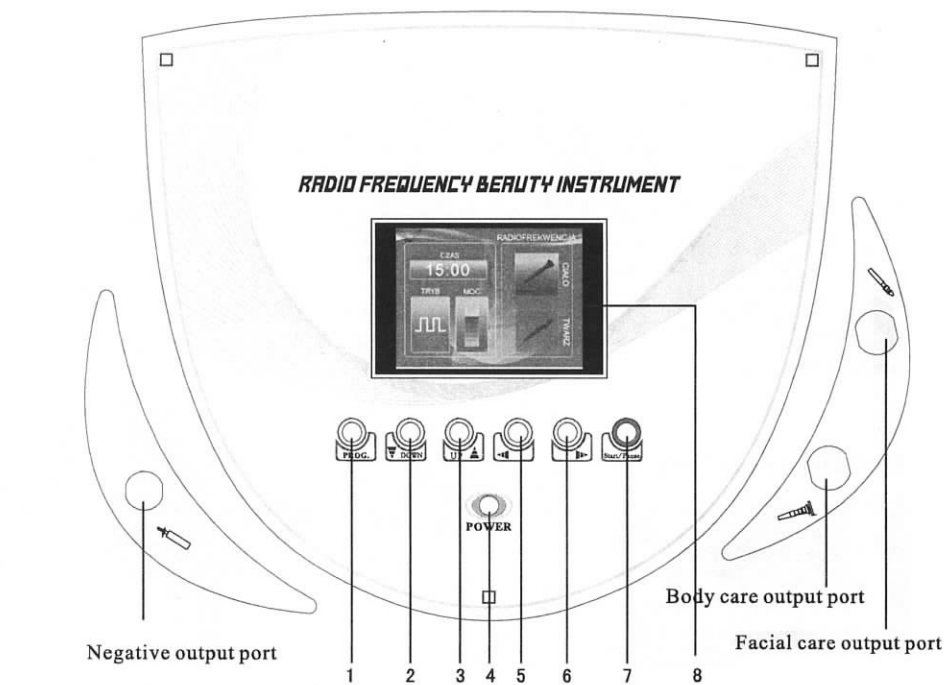

5

- 1.function confirmation
- 2.parameter adjustment “-”
- 3.parameter adjustment “+”
4. Switch of the the machine
- 5-6.swift to other function
- 7.start/pause
- 8.display

Accessory

- 1.Big RF handle
- 2.Small RF handle
- 3.1# treatment head(for body)
- 4.2# treatment head(for body and face)
- 5.3# treatment head(for body and face)
- 6.4# treatment head(for face)
- 7.5# treatment head(for face)
- 8.6#treatment head(for small areas)
- 9.7# treatment head(for eyes)
10. Handhold metal stick

Operation guidance and attention


Operation guidance

- 1.Connect the instrument to power supply
- 2.Connect the treatment heads to the instrument.

Function adjust guidance


1.Time adjustment

- 1) Turn on the mainframe (as picture1)
- 2) Press to select Time, when the mark — is in time, press to adjust time as the following picture;

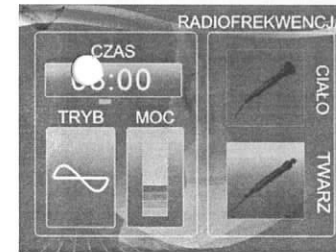

2.Mode adjustment

- 1) Press to select Mode, when the mark — is in Mode, press to adjust the mode as the following picture;

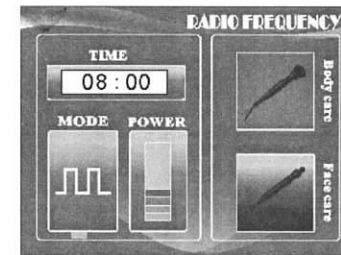



3. Intensity adjustment


- 1) Press to select Power, when the mark — is in Power, press to adjust intensity as the following picture;


6

Operation guidance

4. Function adjustment

please press   the button of switch. if choose 'FACE CARE', as picture 1 it starts to work when the icon becomes red.

Then press  to the next interface as picture 2

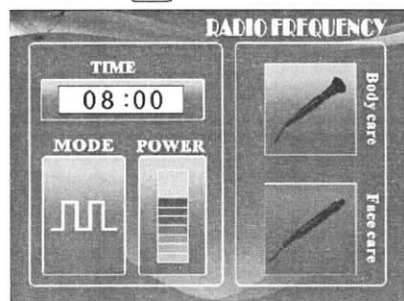

Picture 1




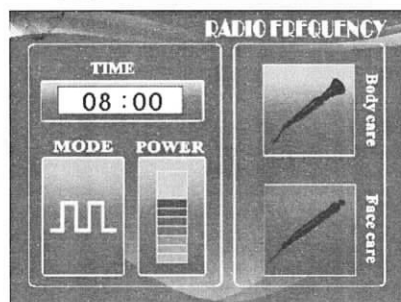
Picture 2

Press   to select Face Lifting as follows



Body Slimming
Face Wrinkle
Face lifting
Buttocks Lifting
Belly Wrinkle

Press  back to previous interface as follows



Note: The parameter is the default face, you can adjust it as previous operation guidance.


Attention

Treatment attention:

1. During operation, because of individual difference, so need to communicate with client and know their feeling, should adjust the intensity according to specific demand.
2. For the treatment of sensitive part, can adjust the intensity to 3 to 5, which is subject to client's specific condition.
3. For the small treatment head, the current is concentrated, the intensity should be adjusted to smaller; For the big treatment head, the current is dispersed, the intensity should be adjusted to bigger.
4. Treatment head should continuous move upon skin, should not stay in a part for more than 3 seconds.
5. If continuous using for half an hour, should stop the operation and rest it for at least 10 minutes for the next operation, in this way can extend the life of Instrument.

Operation guidance

Attention: During treatment, there must be no direct body touch between the client and any other people, the beautician should wear insulating glove, in this way to avoid unnecessary injury.

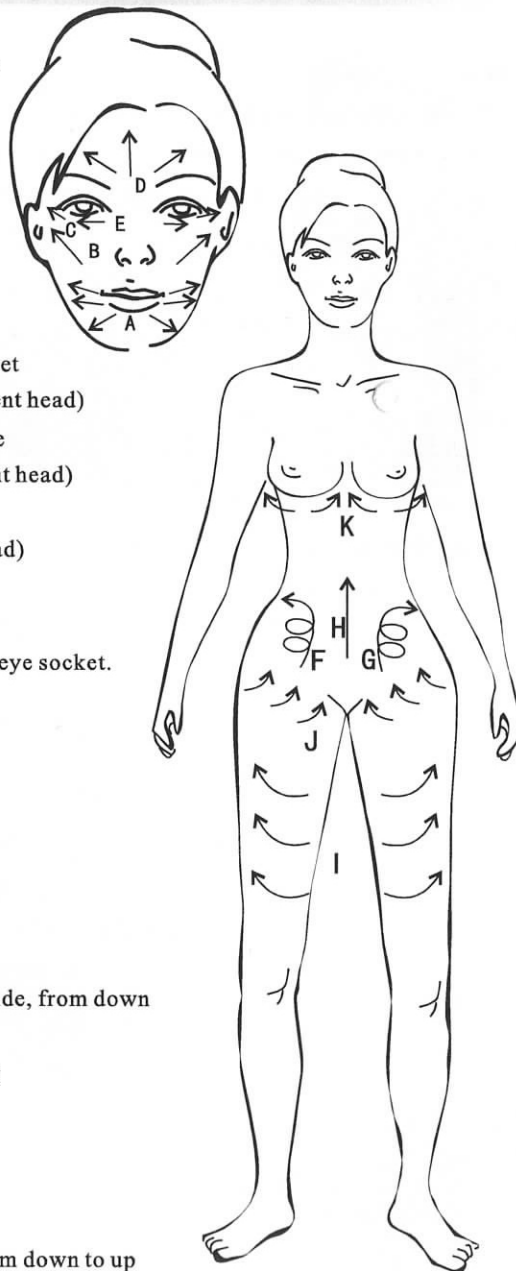

1. Picture guide for face

- A. Lift chin, improve mouth's feet
(Use 2#, 3#, 4#, 5# treatment head)
- B. Lift cheek, improve face loose
(Use 2#, 3#, 4#, 5# treatment head)
- C. Improve crow's feet
(Use 3#, 4#, 5# treatment head)
- D. Improve forehead wrinkle
(Use 2#, 3# treatment head)
- E. Get rid of eye pouch and dark eye socket.
(Use 6#, 7# treatment head)

2. Picture guide for belly

- (Use 1# treatment head)
- F. Clockwise movement
- G. Anticlockwise movement
- H. Movement from down to up

3. Picture guide for thigh

- (Use 1# treatment head)
- I. Movement from inside to outside, from down to up

4. Picture guide for breast lifting

- (Use 1# treatment head)

K. Lift from down to up

5. Picture guide for hips lifting

- (Use 1# treatment head)

J. Lift from outside to inside, from down to up

Technical data and Solution for simple malfunction

●Technical data

Voltage	AC : 110V	You may rest assured that this product is met EN 60335-1 safety standard and has got the CE certificates of EU.
Frequency	60HZ	
Power	200W	
Gross Weight	7.5 kg	
Measurement	55.5X46X28cm	

Note: The design and instruction of this product may be changed without notification.

●Solution for simple malfunction

Malfunction	Solution
Power on, no response	Check whether power is in right connection
	Check whether power switch is on
No output	Check if the probe is good connection with the mainframe
Machine is overheating during operation	Used for too long a time, need to power off and cool in a natural way
	Check if the power supply is in right condition (by technician)
No power	Check whether fuse is broken, if so replace another one

Packing List

Name	Unit	Qty	Remark	
Mainframe	Set	1		
Big RF handle	Piece	1		
Small RF handle	Piece	1		
Metal Stick	Piece	1		
Treatment Heads	Set	1	7 treatment heads	
Power wire	Strip	1		
Fuse	Piece	2		
User manual	Piece	1		

Notice:

1. The instrument has been delivered with full items, when received the instrument, please check all the accessories whether something is missing, and refer to our packing list.
2. If something is broken during transportation, please report to the dealer and send us a copy information.
3. Please reserve the original packing items, when need maintain, it is necessary to send us the instrument with original packing.

Important Safety Notice

- 1 The machine is professional instrument for beauty salon, in order to avoid any unnecessary hurt, need professional beautician to make operation.
2. Do not disassemble the machine or operate the machine that not explained in the instruction book. All the maintenance services shall be carried out by the professional maintenance technician approved by our company.
3. Do not install or operate the machine with wet hands or near the water sources so as to avoid the liquid spraying/swashing over the machine.
4. Pull out the plug and connect with the maintenance service center in the following cases:
 - A. liquid spray/swash over the machine or the machine get caught in rain or water.
 - B. the machine has peculiar smell, fume/smog or abnormal noise.
 - C. power wires damaged or scratched.
 - D. instrument fell down or the shell damaged.
5. Do not put any item on the power wires, or install the instrument in the place where its power wires will be trampled.
6. Pull out the power wires at the end of use for the sake of safety.
7. Do not insert any type of item into the instrument opening; otherwise there will be a fire or electric shock. In case of something fell into the opening but you could not take it out, please pull out the power wires and connect with the dealer or our company.
8. Do not put the instrument on the unstable trolley, shelf or vehicle, if fall down, the instrument may be damaged.
9. To guarantee the life circle of this machine, the interval of the continuous switch on and off should be over one minute.
10. The power shall comply with the specification specified in the label, otherwise the machine will not run or run abnormally, even be burnt out.
11. The company has right to revise or amend this manual.

The company reserves all the rights of interpreting the information mentioned above.